



GRIFFIN DENTAL  
LABORATORIES

Get More **INFORMATION**  
Call Now!

**1-859-554-8410**



**Big Lab Technology Meets Small Lab Personalized Service**

# Full Contour Zirconia

by **Griffin Dental Laboratories**



Griffin Dental Laboratories produces zirconia restorations in house. The restorations have an extremely high bending strength, optimal edge stability in the restorations fabricated with this technology.

## Prep Requirements

- Preparation suitable for ceramics, i.e no sharp corners, edges, and internal corners.
- Circular chamfer with cervical edge of at least 0.8mm
- Circular shoulder with rounder internal corners
- Inscal and occlusal reduction for faced crowns: 1.5mm to 2.0mm
- Minimum stump height 3.0mm

## Contraindications

- Deep Cavities in the occlusal area.
- Rising chamfers (gutter preparations)
- Finished margins

## Cementation

Conventional Cementation: Due to the high degree of rigidity and stability of the zirconia framework conventional fixing with

- Zinc oxide phosphate cement or
- Glass ionomer luting (i.e. GC Fuji Plus) is possible in most cases.

With convention cements make sure there is sufficient retention that the stump height is 3 mm.

Adhesive fixing: For adhesive fixing we recommend the fixing composites Panavia® 21 or Panavia® F 2.0 due to their excellent adhesion with the zirconium oxide framework material.

Sales Representative:



GRIFFIN DENTAL  
LABORATORIES

Type
<b>NEW!</b> Full Contour Zirconia
Porcelain Fused-to-Zirconia
*For large round house Cases call for Pricing

[www.GriffinDentalLaboratories.com](http://www.GriffinDentalLaboratories.com)

651 Perimeter Drive, STE 600 Lexington, KY 40517 1-859-554-8410

©2020 Griffin Dental Laboratories, LLC . All Rights Reserved.